

Skin Check MSS200

With the **Skin Check - MSS200** dermatologists and operators of tanning studios get a compact device which allows them to determine the skin type of a person within seconds.

Whilst the classification of the skin type has been more or less subjective up to now, with the **Skin Check - MSS200** it is now possible to make an optimal evaluation of the skin type.

Tanning studio owners and users can now optimally adjust the tanning duration to the respective skin type. The risk of improper use and the hereby resulting hazard of sunburn or a non satisfying tanning are herewith reduced.

For all dermatologists the **Skin Check - MSS200** is a natural help to align a necessary treatment directly to the skin type

The **Skin Check - MSS200** advantage is the easy, clear and user guided installation and operation. The input of all parameters is done via a menu. False input is hereby reduced to a minimum.

Skin Check MSS200



Skin Check - MSS200 connect via USB to your PC

Skin Check - MSS200 Software



The delivery contains:

- ▶ Skin Check - MSS200
- ▶ USB-adaptor cable
- ▶ Calibration probe
- ▶ Evaluation software
- ▶ User manual

Skin Check MSS200

Recommended Exposure Schedule

RECOMMENDED EXPOSURE SCHEDULE		MAXIMUM EXPOSURE TIMES (in minutes)			
CLASS	SKIN TYPES	week 1	week 2	week 3	week 4
I	Sensitive Skin (Burns easily and severely, and does not tan)	TANNING NOT ADVISED			
II	Light Skin (Burns easily and severely, and tans minimally)	10	15	20	30
III	Normal Skin (Burns moderately, and tans moderately)	12	18	23	30
IV	Dark Skin (Burns minimally, and tans well or above average)	15	22	27	30

YOUR SKIN TYPE: 2.4

	8	12	16	24
Excellence	8	12	16	24
Classic 200	6	10	13	19
Classic 300	5	9	11	16
Crye Pro	5	7	9	14
Lumina	4	6	8	12
Advantage	4	5	7	11
Evolution	3	5	6	10
Open Sun	3	4	6	9
Sprint	3	4	5	8
MegaSun	2	4	5	7

DANGER :
 Follow proper operation instructions from the operator. When using a machine with bench and canopy, put the canopy down until it will move no further. Operation of any tanning device to the body has the recommended minimum distance for use may result in overexposure. Recommended eyewear. Avoids eye contact. Such as goggles.

WARNING :
 In 2011 IEC equipment conforms to standard IEC 61010-20.
 2011 IEC part of protective equipment is furnished with this equipment and should be used by the operator. The user manual and lamp are included. **THE LAMP USED CONTAINS MERCURY. USE CAREFULLY TO AVOID CONTACT WITH THE LAMP.**
 ALL LAMPS MUST HAVE LABELS. STAY IN COMPLIANCE WITH PERFORMANCE STANDARDS FOR SUN LAMP PRODUCTS UNDER IEC PART 1040-20.

Customer: _____ Tanning Center: _____ Date: _____
 Address: _____ MSS Elektronik GmbH
 _____ Tel. +49-2378-913695
 _____ Schlesierstr. 5a
 _____ D-58730 Fröndenberg

Signature _____ Signature _____

The user manual describes all important information like for instance:

- Installation
- Operation of the evaluation software
- Determination of the individual skin type
- Evaluation of the measurement
- General advices, maintenance as well as technical data

Subject to technical changes.